

## SHORTCUT SHEET

### DIGI VOX 2



The DigiVox 2 is a digitized augmentative communication device. It accommodates up to 142 minutes of recording time in 48 levels and 6 user groups. It comes with a set of 18 overlays that can be labeled with any combination of symbols or letters. Single words, phrases and sentences can be programmed. The methods of access include direct selection, visual and auditory scanning.

#### TO USE

1. To Program:
  - a. Put the DigiVox in **Program Mode**.
  - b. Press **Set Up**.
  - c. Press **Program/Play**.
  - d. Press **Up/Down Arrow** until display reads Program.
  - e. Press **OK**.
  - f. Select the level to program by pressing the **Level** key until the desired level is reached (1 through 48).
  - g. Choose **Keyboard Layout**.
  - h. Press **Set Up**.
  - i. Press **Keyboard Layout**.
  - j. Press **Up/Down Arrow** until the desired layout is reached.  
Sizes vary from 1x2 (2 keys) to 4x12 (48 keys).
  - k. Press **OK**.
  
2. To Record a Message:
  - a. Press **Record**.
  - b. Display will read OPress and Hold a key to record upon O.
  - c. Press and hold the **key** as the message is spoken into the microphone.
  - d. Digivox will beep twice indicating that the message is recorded.
  - e. To review the message, press the **key** that was recorded on.
  
3. Put the Digivox in **Play Mode** before using.

- a. Press **Set Up**.
- b. Press **Program/Play**.
- c. Press **Up/Down Arrow** until display reads **Play**.
- d. Press **OK**.

## PRODUCT INFORMATION

**Contact Information:** DynaVox Systems, Inc.  
2100 Wharton St.  
Pittsburgh, PA 15203  
1-888-697-7332  
www.dynavoxsys.com  
**(Product has been discontinued)**

*Edited by: Tammy Sproule, South-Central BOCES SWAAAC Coordinator, May 15, 2004*

For more SWAAAC Shortcut Sheets, go to [www.swaaac.org](http://www.swaaac.org), click on **AT Loan Bank** and then click on **Shortcut Sheets**.  
Would you like a Shortcut Sheet that doesn't exist? Contact the WestAC office at (970) 248-0876 or  
[carrie.foster@uchsc.edu](mailto:carrie.foster@uchsc.edu).