

SHORTCUT SHEET

ENKIDU HANDHELD IMPACT



The Handheld is a portable Windows CE computer running the Impact communication software. The Handheld Impact combines a large keyboard with a touch screen to provide additional methods of message production. The device comes installed with 10 standard configurations.

TO USE

TO CREATE A CONFIGURATION

A configuration is a set of vocabulary pages. Typically, all of an individual's pages will be in one configuration.

Create a new configuration using a Standard Configuration as a starting point:

1. Go to **File Menu**.
2. Choose **Standard Configuration**.
3. Choose the configuration you would like to start with.
4. To Save your new configuration:
 - a. Go to **File Menu**
 - b. Choose **Configuration**
 - c. Choose **Save Configuration As**
 - d. Give your configuration a name

Create a new blank configuration:

1. Go to **File Menu**
2. Choose **Configuration**
3. Choose **New Configuration**
4. Type a name for the page
5. Choose the **Arrow** next to **Page Type**
6. Choose the Page Layout that you prefer for this page

TO CREATE A BUTTON THAT SPEAKS

1. Go to **View Menu**

2. Choose **Go To Edit Mode**
3. Choose the **Page To Modify**
4. Double click on **Button You Want To Speak**
5. Choose **Speak Label** from the Action Box
6. Type the label for the button
7. Choose **Browse** and then a **Picture**
8. Click **OK**
9. Go to **View Menu**
10. Choose **Go to Run Mode**

TO CREATE A BUTTON THAT SPEAKS MORE THAN ONE LABEL

1. Go to **View Menu**
2. Choose **Go To Edit Mode**
3. Choose the **Page to Modify**
4. Double click on the **Button you want to speak**
5. Choose **Speak Message** from the Action Box
6. Type the message for the button
7. Type a label for the button
8. Choose **Browse** and choose a **Picture**
9. Click **OK**
10. Go to **View Menu**
11. Choose **Go to Run Mode**

TO CHANGE A BUTTON'S PICTURE

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **Page to Modify**
4. Double click on the **Button you want to speak**
5. Choose **Browse** and then choose a **new picture**
6. Click **OK**
7. Go to **View Menu**
8. Choose **Go to Run Mode**

TO CREATE A PAGE

Pages are levels of specific vocabulary linked together to create a vocabulary set specific to the device user.

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Go to **View Menu** again
4. Choose **Page Editing**
5. Choose **Add Page** (NOTE: There is also an ADD PAGE shortcut icon at the top of the screen when you are in Edit Mode. Choosing this icon will also create a new blank page).
6. Type a name for the page
7. Choose the **Arrow** next to **Page Type** and choose the **Page Layout** that you prefer.
8. Be sure to add a **Navigation button** to your page or you will not be able to navigate to or from this page.

TO ADD A NAVIGATION BUTTON TO A PAGE

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **page to modify**
4. Double click on the button you want to be your navigation button
5. Choose **Navigate** or **Visit** from the Action Box
6. (NOTE: Navigate will take you to a page and leave you there, while Visit will take you there for one selection and immediately return to the previous page).
7. Choose the **page** that you want to link to from the Navigate/Visit box
8. Type a **label** for the button
9. Choose **Browse** and choose a **Picture** that you want to represent your navigation button
10. Click **OK**
11. Go to **View Menu**
12. Choose **Go to Run Mode**

TO ADD A NAVIGATION BUTTON THAT ALSO SPEAKS

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **page to modify**
4. Double click on the button you want to be your navigation button
5. Choose **Speak Message** from the Action Box
6. Type the message you want spoken into the Message Box
7. Choose **Navigate or Visit** from the Second Action Box
8. Choose the **page** that you want to link to from the Navigate/Visit Box
9. Type a **label** for the button
10. Choose **Browse** and choose a **picture** that you want to represent your navigation button
11. Tap **OK**
12. Go to **View Menu**
13. Choose **Go to Run Mode**

TO CREATE A BUTTON TO PERFORM A STANDARD FUNCTION

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **Page to Modify**
4. Double click on the **Button you want to speak**
5. Choose **Control** from the Action Box
6. Choose the **Function** that you want for the button to perform from the Function Box
7. Click **OK**
8. Go to **View Menu**
9. Choose **Go to Run Mode**

TO MODIFY BUTTON APPEARANCE

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **Page to Modify**
4. Double click on a **Button**
 - a. To modify colors on a specific button, choose the This **button** tab and

- make your changes
- b. To modify colors, the font, or style on all buttons, choose the **All Buttons** tab and make your changes.
 - c. To modify the page background color, choose the **This Page** tab and choose the color of your choice
5. Click **OK**
 6. Go to **View Menu**
 7. Choose **Go to Run Mode**

TO MODIFY THE PRONUNCIATION OF A WORD

1. Go to **Tools**
2. Choose **Pronunciation Editor**
3. Choose **Add**
4. Type the word
5. Type the words proper pronunciation
6. Test by choosing **Try Pronunciation**
7. Choose **OK**

PRODUCT INFORMATION

Contact Information: Enkidu Research
2100 Wharton Street
Suite 400
Pittsburgh, PA 15203
(800) 344-1778
www.enkidu.net

Cost: \$3995

Created By: Debby McBride, AAC Connections, 04-04

Edited By: Tammy Sproule, South-Central BOCES, SWAAAC Coordinator, 04-01-04

For more SWAAAC Shortcut Sheets, go to www.swaaac.org, click on **AT Loan Bank** and then click on **Shortcut Sheets**.
Would you like a Shortcut Sheet that doesn't exist? Contact the WestAC office at (970) 248-0876 or
carrie.foster@uchsc.edu.