

## SHORTCUT SHEET

### ENKIDU PALMTOP PORTABLE IMPACT



The PalmTop Portable Impact is the smallest of the Enkidu Impact Communication devices. It uses digitized and synthesized speech. The PalmTop Portable IMPACT combines a keyboard with a touch screen to provide additional methods of message production. The device comes installed with 10 standard configurations.

#### TO CREATE A CONFIGURATION

A configuration is a set of vocabulary pages. Typically, all of an individual's pages will be in one configuration.

1. To create a new configuration using a Standard configuration as a starting point:
  - a. Go to **File Menu**.
  - b. Choose Standard Configurations.
  - c. Choose the configuration you would like to start with.
2. To Save your new configuration:
  - a. Go to **File Menu**.
  - b. Choose **Configuration**.
  - c. Choose **Save Configuration As**
  - d. Give your configuration a name
3. Create a new blank configuration
  - a. Go to **File Menu**
  - b. Choose **Configurations**
  - c. Choose **New Configurations**
  - d. Type a name for the page
  - e. Choose the **Arrow next to Page Type**
  - f. Choose the **Page Layout** that you prefer for this page.

#### TO CREATE A BUTTON THAT SPEAKS

1. Go to **View Menu**
2. Choose **Go To Edit Mode**
3. Choose the **Page you want to Modify**
4. Double click on **Button you want to Speak**
5. Choose **Speak Label** from the Action Box

6. Type the label for the button
7. Choose **Browse** and then a **Picture**
8. Tap **OK**
9. Go to **View Menu**
10. Choose **Go to Run Mode**

#### **TO CREATE A BUTTON THAT SPEAKS MORE THAN JUST THE LABEL**

1. Go to **View Menu**
2. Choose **Go To Edit Mode**
3. Choose the **Page to Modify**
4. Double click on the **Button you want to speak**
5. Choose **Speak Message** from the Action Box
6. Type the message for the button
7. Type a label for the button
8. Choose **Browse** and choose a **Picture**
9. Tap **OK**
10. Go to **View Menu**
11. Choose **Go to Run Mode**

#### **TO CHANGE A BUTTON'S PICTURE**

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **Page to Modify**
4. Double click on the **Button** you want to speak
5. Choose **Browse** and then choose a new picture
6. Tap **OK**
7. Go to **View Menu**
8. Choose **Go to Run Mode**

#### **TO CREATE A PAGE**

Pages are levels of specific vocabulary linked together to create a vocabulary set specific to the device user.

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Go to **View Menu** again
4. Choose **Page Editing**
5. Choose **Add Page**  
(NOTE: There is also an ADD PAGE shortcut icon at the top of the screen when you are in Edit Mode. Choosing this icon will also create a new blank page).
6. Type a **name** for the page
7. Choose the **arrow** next to **Page Type** and choose the **page Layout** that you prefer.
8. Be sure to add a **Navigation button** to your page or you will not be able to navigate to or from this page.

## TO ADD A NAVIGATION BUTTON TO A PAGE

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **page to modify**
4. Double click on the button you want to be your navigation button
5. Choose **Navigate** or **Visit** from the Action Box (NOTE: Navigate will take you to a page and leave you there, while visit will take you there for one selection and immediately return to the previous page).
6. Choose the **page** that you want to link to from the Navigate/Visit box
7. Type a **label** for the button
8. Choose **Browse** and choose a **picture** that you want to represent your navigation button
9. Tap **OK**
10. Go to **View Menu**
11. Choose **Go to Run Mode**

## TO ADD A NAVIGATION BUTTON THAT ALSO SPEAKS

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the page to modify
4. Double click on the button you want to be your navigation button
5. Choose **Speak Message** from the Action Box
6. Type the message you want spoken into the Message Box
7. Choose **Navigate** or **Visit** from the Second Action Box
8. Choose the **page** that you want to link to from the Navigate/Visit Box
9. Type a **label** for the button
10. Choose **Browse** and choose a **picture** that you want to represent your navigation button
11. Tap **OK**
12. Go to **View Menu**
13. Choose **Go to Run Mode**

## TO CREATE A BUTTON TO PERFORM A STANDARD FUNCTION

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the page to modify
4. Double click on the **button you want to speak**
5. Choose **Control** from the Action Box
6. Choose the **function** that you want for the button to perform from the Function Box
7. Tap **OK**
8. Go to **View Menu**
9. Choose **Go to Run Mode**

## TO MODIFY BUTTON APPEARANCE

1. Go to **View Menu**
2. Choose **Go to Edit Mode**

3. Choose the **page** you want to modify
4. Double click on a **button**
  - a. To modify colors on a specific button, choose the **This button** tab and make your changes
  - b. To modify colors, the font, or style on all buttons, choose the **All buttons** tab and make your changes
  - c. To modify the page background color, choose the **This page** tab and choose the color of your choice
5. Tap **OK**
6. Go to **View Menu**
7. Choose Go to **Run Mode**

### TO MODIFY THE PRONUNCIATION OF A WORD

1. Go to **Tools**
2. Choose **Pronunciation Editor**
3. Choose **Add**
4. Type the word
5. Type the words proper pronunciation
6. Test by choosing **Try Pronunciation**
7. Choose **OK**

### PRODUCT INFORMATION

**Contact Information:** Enkidu Research  
 247 Pine Hill Road  
 Spencerport, NY 14559  
 (800) 273-6315  
 www.enkidu.net

**Cost:** \$2950 to \$3995

*By: Debby McBride, AAC Connections, 04-04*

*Edited By: Tammy Sproule, South-Central BOCES, SWAAAC Coordinator, 04-01-04*

For more SWAAAC Shortcut Sheets, go to [www.swaaac.org](http://www.swaaac.org), click on **AT Loan Bank** and then click on **Shortcut Sheets**.  
 Would you like a Shortcut Sheet that doesn't exist? Contact the WesTAC office at (970) 248-0876 or [carrie.foster@uchsc.edu](mailto:carrie.foster@uchsc.edu).