

## SHORTCUT SHEET

### ENKIDU TABLET IMPACT



The Tablet Impact is the largest of the Enkidu Impact Communication devices. It uses digitized and synthesized speech. The Tablet combines a keyboard with a large touch screen to provide additional methods of message production. With a full sized screen, the Tablet offers even more space for button, making it ideal for phrase and symbol based communication pages. The device comes installed with 10 standard configurations.

#### TO USE

##### **TO CREATE A CONFIGURATION**

A configuration is a set of vocabulary pages. Typically, all of an individual's pages will be in one configuration.

Create a new configuration using a Standard Configuration as a starting point:

1. Go to **File Menu**.
2. Choose **Standard Configuration**.
3. Choose the configuration you would like to start with.
4. To Save your new configuration:
  - a. Go to **File Menu**
  - b. Choose **Configuration**
  - c. Choose **Save Configuration As**
  - d. Give your configuration a name

Create a new blank configuration:

1. Go to **File Menu**
2. Choose **Configuration**
3. Choose **New Configuration**
4. Type a name for the page
5. Choose the **Arrow** next to **Page Type**
6. Choose the Page Layout that you prefer for this page

##### **TO CREATE A BUTTON THAT SPEAKS**

1. Go to **View Menu**

2. Choose **Go To Edit Mode**
3. Choose the **Page To Modify**
4. Double click on **Button You Want To Speak**
5. Choose **Speak Label** from the Action Box
6. Type the label for the button
7. Choose **Browse** and then a **Picture**
8. Click **OK**
9. Go to **View Menu**
10. Choose **Go to Run Mode**

#### **TO CREATE A BUTTON THAT SPEAKS MORE THAN ONE LABEL**

1. Go to **View Menu**
2. Choose **Go To Edit Mode**
3. Choose the **Page to Modify**
4. Double click on the **Button you want to speak**
5. Choose **Speak Message** from the Action Box
6. Type the message for the button
7. Type a label for the button
8. Choose **Browse** and choose a **Picture**
9. Click **OK**
10. Go to **View Menu**
11. Choose **Go to Run Mode**

#### **TO CHANGE A BUTTON'S PICTURE**

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **Page to Modify**
4. Double click on the **Button you want to speak**
5. Choose **Browse** and then choose a **new picture**
6. Click **OK**
7. Go to **View Menu**
8. Choose **Go to Run Mode**

#### **TO CREATE A PAGE**

Pages are levels of specific vocabulary linked together to create a vocabulary set specific to the device user.

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Go to **View Menu** again
4. Choose **Page Editing**
5. Choose **Add Page** (NOTE: There is also an ADD PAGE shortcut icon at the top of the screen when you are in Edit Mode. Choosing this icon will also create a new blank page).
6. Type a name for the page
7. Choose the **Arrow** next to **Page Type** and choose the **Page Layout** that you prefer.
8. Be sure to add a **Navigation button** to your page or you will not be able to navigate to or from this page.

### TO ADD A NAVIGATION BUTTON TO A PAGE

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **page to modify**
4. Double click on the button you want to be your navigation button
5. Choose **Navigate** or **Visit** from the Action Box
6. (NOTE: Navigate will take you to a page and leave you there, while Visit will take you there for one selection and immediately return to the previous page).
7. Choose the **page** that you want to link to from the Navigate/Visit box
8. Type a **label** for the button
9. Choose **Browse** and choose a **Picture** that you want to represent your navigation button
10. Click **OK**
11. Go to **View Menu**
12. Choose **Go to Run Mode**

### TO ADD A NAVIGATION BUTTON THAT ALSO SPEAKS

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **page to modify**
4. Double click on the button you want to be your navigation button
5. Choose **Speak Message** from the Action Box
6. Type the message you want spoken into the Message Box
7. Choose **Navigate or Visit** from the Second Action Box
8. Choose the **page** that you want to link to from the Navigate/Visit Box
9. Type a **label** for the button
10. Choose **Browse** and choose a **picture** that you want to represent your navigation button
11. Tap **OK**
12. Go to **View Menu**
13. Choose **Go to Run Mode**

### TO CREATE A BUTTON TO PERFORM A STANDARD FUNCTION

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **Page to Modify**
4. Double click on the **Button you want to speak**
5. Choose **Control** from the Action Box
6. Choose the **Function** that you want for the button to perform from the Function Box
7. Click **OK**
8. Go to **View Menu**
9. Choose **Go to Run Mode**

### TO MODIFY BUTTON APPEARANCE

1. Go to **View Menu**
2. Choose **Go to Edit Mode**
3. Choose the **Page to Modify**
4. Double click on a **Button**
  - a. To modify colors on a specific button, choose the This **button** tab and

- make your changes
- b. To modify colors, the font, or style on all buttons, choose the **All Buttons** tab and make your changes.
  - c. To modify the page background color, choose the **This Page** tab and choose the color of your choice
5. Click **OK**
  6. Go to **View Menu**
  7. Choose **Go to Run Mode**

#### **TO MODIFY THE PRONUNCIATION OF A WORD**

1. Go to **Tools**
2. Choose **Pronunciation Editor**
3. Choose **Add**
4. Type the word
5. Type the words proper pronunciation
6. Test by choosing **Try Pronunciation**
7. Choose **OK**

#### **PRODUCT INFORMATION**

**Contact Information:** Enkidu Research  
2100 Wharton Street  
Suite 400  
Pittsburgh, PA 15203  
(800) 344-1778  
[www.enkidu.net](http://www.enkidu.net)

**Cost:** \$4695

*Created By: Debby McBride, AAC Connections, 04-04*

*Edited By: Tammy Sproule, South-Central BOCES, SWAAAC Coordinator, 04-01-04*

For more SWAAAC Shortcut Sheets, go to [www.swaaac.org](http://www.swaaac.org), click on **AT Loan Bank** and then click on **Shortcut Sheets**.  
Would you like a Shortcut Sheet that doesn't exist? Contact the WestTAC office at (970) 248-0876 or [carrie.foster@uchsc.edu](mailto:carrie.foster@uchsc.edu).