

SHORTCUT SHEET

MESSAGEMATE



MessageMates are extremely rugged hand-held communication devices that allow the user to speak a variety of recorded messages simply by pressing a key. There are four different MessageMate models to choose from: Multi-Level MessageMate 40, MessageMate 40, MessageMate 20, and Mini-MessageMate. Each model is available with a variety of recording times and comes with a comprehensive user's manual, an AC adapter, and a long-lasting rechargeable battery.

TO USE

1. Turn the MessageMate on using the dial on the **Right End Panel** of the device. This dial also adjusts the volume.
2. Press each **Message Key** to listen to each message.

TO RECORD

1. Make sure that the Record Lock/Unlock switch on the Right End Panel is in the **unlocked** position.
2. Press and hold the **Record key** on the top of the device while holding down the **message key** that you want selected. The message and Record key LED lights will come on.
3. Speak your message into the MIC while the message LED light is on.
4. Release and check that it worked by pressing the **message key**.

LEFT END PANEL CONTROLS

The Left End Panel includes a Scan Rate Adjustment dial (if equipped) and DIP Switch Options – a set of numbered switches. Using the directions provided on the back of the MessageMate, you can select the type of scan (horizontal, linear, auditory), then number of message keys, the Accept Time required to activate the message, and the Scan Rate.

SCANNING

When using the visual scan feature of the MessageMate, be sure that the message overlay is printed on transparency film so that the messageLED lights can be seen.

PRODUCT INFORMATION

Contact Information: Words+, Inc.
1220 W. Avenue J
Lancaster, CA 93534-2902
(800) 869-8521
www.words-plus.com

Pricing:

MessageMate 40/600	10 minutes	\$1349.00
MessageMate 40/300	5 minutes	\$1149.00
MessageMate 40/150	2.5 minutes	\$949.00
MessageMate 20/150	2.5 minutes	\$799.00
MessageMate 20/75	75 seconds	\$599.00
MessageMate 8/60	60 seconds	\$399.00

Created by Tony Arnt, Assistive Technology Partners, 4/19/2004

Edited by Carrie Foster, Assistive Technology Partners/WesTAC, 5/20/04

For more SWAAC Shortcut Sheets, go to www.swaac.org, click on **AT Loan Bank** and then click on **Shortcut Sheets**. Would you like a Shortcut Sheet that doesn't exist? Contact the WesTAC office at (970) 248-0876 or carrie.foster@uchsc.edu.